

Instructions for Maintenance and Cleaning of Blood Transport Boxes

Cleaning

If required following inspection clean all elements of Transport Stabilisation Systems with Ready-to-use Sprays or wipes

- Allow 5 minutes contact time
- Ensure thoroughly dry before returning to use.

Short Journey Va-Q-Tec Transport Box and Stabilisation System

Container – Outer Textile Bag

Outer	Cordura	<p>Check the bag for damage and/or soiling</p> <ul style="list-style-type: none"> • Material surfaces • Handles • Closing mechanism • Pouches / Pockets • Plastic 'shoe'
-------	---------	---

Container – Insulated Inner

Outer	EPP (Expanded Polypropylene) and PP (Polypropylene)	<p>The following must be checked in detail for visible damage such as holes or breaks</p> <ul style="list-style-type: none"> • The outer wall of the container • The inner walls of the container
Inner and Lid	EPP (Expanded Polypropylene) and PP (Polypropylene)	<p>The following must be checked in detail for visible damage such as holes or breaks</p> <ul style="list-style-type: none"> • The outer wall of the lid • The inner walls of the lid • The top of the lid • The bottom of the lid • The lid lip